

Gravity Feeding Sets with ENFit®

Gravity Feed Sets	Item #	Units/Case
40mm Screw Cap Set with ENFit	VED-022	30
ENPlus Spike with ENFit	VED-028	30
500mL Bag Set (LL)* with ENFit	VED-040	30
500mL Bag Set (PVC) with ENFit	VED-042	30
1000mL Bag Set (LL) with ENFit	VED-034	30
1000mL TOP FILL Bag Set (PVC) with ENFit	VED-046	30

Setting the Feed Rate

- Vesco Medical utilizes the same drip/site chamber across all gravity items that feature 14 drops per mL. **Formula: (Drops/Minute) X 4.26 = Feeding Rate (mL/Hr)**

Feeding Rates		
Feeding Rate (mL/Hr)	Drops/Minute	Seconds Between Drops
5	1	52.1
10	3	26
15	4	17.3
20	5	13.9
25	6	10.3
30	7	8.6
50	12	5.1
65	15	4.0
75	18	3.4
100	23	2.6
150	35	1.7
200	47	1.3
250	58	1.0

Important Safety Information

- To avoid risk of a tubing misconnection, **ONLY** connect the distal connector to an ENTERAL feeding tube. This device is compatible with a flexible female connector on the patient's enteral feeding tube.
- Not for IV use.
- This device should be replaced at least every 24 hours.
- Do not store for prolonged periods at extreme temperatures.
- See individual product labeling for directions and important safety information.
- Single patient use.

