




## Gravity Feed Set with ENFit® Instructions for Use

**WARNING: FOR ENTERAL FEEDING ONLY.** Devices used in breathing systems & driving gases, urethral / urinary, limb cuff inflation, neuraxial and intravascular/ hypodermic applications may have the potential for misconnection. This device is intended to deliver liquid nutritional formulas or water to a patient's enteral access device (feeding tube).

### DIRECTIONS FOR ENTERAL FEEDING:

1. Confirm enteral feeding tube is properly placed in patient and is not clogged.
2. Complete a patient label and affix to feed set.
3. Close feeding-set roller clamp.
4. A) Fill bag with nutrition and close bag cap securely.  
B) Connect Screw Cap to nutrition container. Turn clockwise until securely fastened.
5. Suspend container at least 20 inches above patient's head.
6. Squeeze feed-set sight chamber until 1/3 to 1/2 full.
7. Remove cover from ENFit connector. Open roller clamp and allow fluid to expel air from tubing then close roller clamp.
8. Attach female ENFit from the gravity feed set to ENFit male on feeding tube.
9. Set delivers approximately 14 drops/milliliter. Adjust roller clamp to control flow rate.

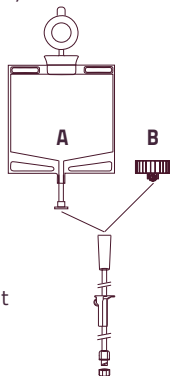

### PRECAUTIONS:

- ONLY connect distal connector to an ENTERAL feeding tube. This device is compatible with a male ENFit connector on the patient's enteral feeding tube.
- This device should be replaced at least every 24 hours.
- Do not store for prolonged periods of time at extreme temperatures.
- For single patient use only.

**Vesco Medical, LLC**  
60 Collegeview Rd., Westerville, OH 43081  
www.vescomedical.com; 855-239-0131  
© 2023 Vesco Medical, LLC; LBL 071-03-01  
Made in Dominican Republic.  
ENFit is a registered trademark of GEDSA.



Prescription  
use only



Single  
Patient Use



Not for  
IV Use



NON  
STERILE  
Non-sterile



Not made with  
natural rubber latex



Not made  
with DEHP